

How to Run a Sample Using the InSight V3 PLUS

1. Use EDTA K3 anticoagulated blood collection tube
2. Fill 2/3's of the tube with blood
3. Ensure a background count and auto clean has been performed before analysis
4. Use the touch screen to select 'Test' and then select 'Species' to set up the correct species for testing
5. Select 'Next Sample'
6. Enter Patient details, press 'OK' to save
7. Thoroughly mix blood on roller mixer or by slowly inverting the tube 10 times
7. REMOVE SAMPLE CAP and place sample under the probe
8. Push the sampling trigger
9. Remove the sample when the analyser 'beeps'
9. Results will automatically display and print out once analysis is complete



If you require assistance, please contact your Woodley Equipment Area Account Manager or
Woodley Technical Support Dept. on +44 (0)1204 669033

Tel: +44 (0)1204 669033
Fax: +44 (0)1204 669034
Email: sales@woodleyequipment.com
Web www.woodleyequipment.com

WOODLEY
EQUIPMENT COMPANY LTD.

OLD STATION PARK BUILDINGS . ST. JOHNS STREET . HORWICH . BOLTON . LANCASHIRE . BL6 7NY