





How to Run a Sample


Switch On and Login

1. Turn on EPOC Reader & EPOC Host
2. Login to Host
3. Connect Reader to Host by selecting icon

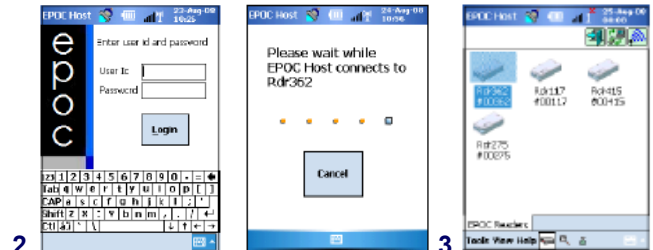

Start the Test

4. Remove card from pouch
5. Firmly slide card into the Reader blue side facing upwards, calibration will commence- duration 165 secs
6. Enter patient ID / information
7. Take blood sample from patient- 100µl required**
(**alternatively take blood sample prior to point 5 if preferred)
8. When EPOC displays 'Calibration Finished. Insert Sample...' introduce sample within 5 minutes
9. Hold syringe barrel vertically
10. Place syringe into blood sample entry port and rotate ¼ turn to secure
11. Steadily depress syringe plunger in a single smooth continuous motion until audible beep sounds
12. Read results in 30 seconds & select  icon to print
13. Remove card and discard in clinical waste

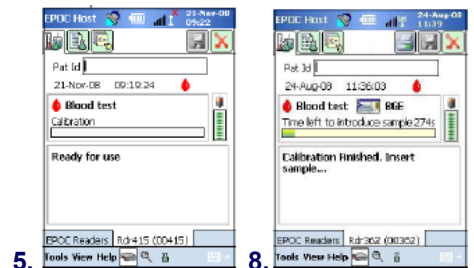

11.

If you require assistance please contact your Woodley Equipment Area Account Manager or Woodley Technical Support Dept. on 01204 669033

Tel: +44 (0)1204 669033
Fax: +44 (0)1204 669034
Email: sales@woodleyequipment.com
Web www.woodleyequipment.com

WOODLEY
EQUIPMENT COMPANY LTD.

OLD STATION PARK BUILDINGS . ST. JOHNS STREET . HORWICH . BOLTON . LANCASHIRE . BL6 7NY