


Overview


These instructions describe how to upgrade the ePOC Host software using the EDM.

How to Load an Upgrade File into the EDM

1. Save the desired Host upgrade file to disk. (i.e. **c:\Temp**).
2. Using Internet Explorer, connect to the EDM (ie. <http://localhost/EpocManager>) and log in as administrator. (User Name: **epocsysadmin**, Password: **epocadmin**)
3. From the EDM menu, select **EPOC System**→**Host settings**→**Software update**.

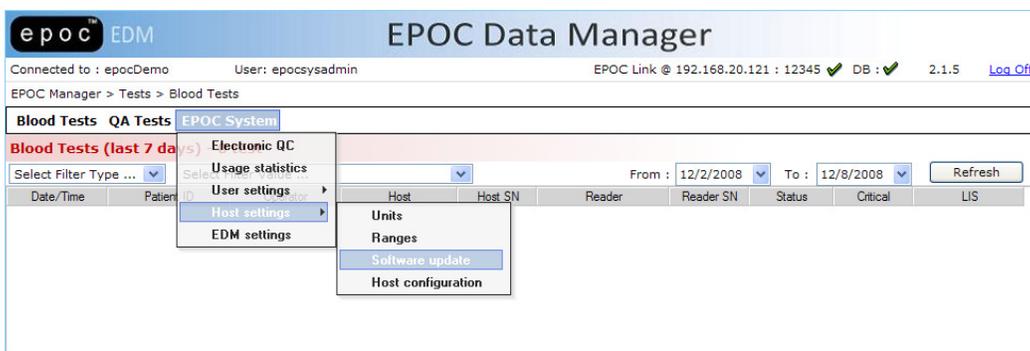

4. Click **Browse...** and select the upgrade file (i.e. **c:\Temp\epocHostUpgrade_v#.##**).

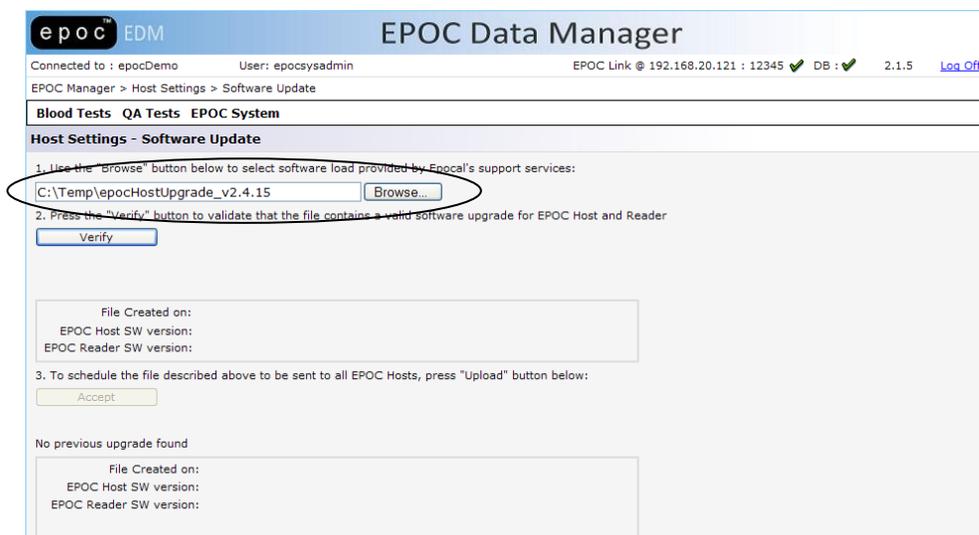

5. Click **Verify**. The upgrade information will appear in the box below the Verify button.

epoc[™] EDM EPOC Data Manager
Connected to : epocDemo User: epocsysadmin EPOC Link @ 192.168.20.121 : 12345 ✓ DB : ✓ 2.1.5 [Log Off](#)
EPOC Manager > Host Settings > Software Update
Blood Tests QA Tests EPOC System
Host Settings - Software Update
1. Use the "Browse" button below to select software load provided by Epocal's support services:

2. Press the "Verify" button to validate that the file contains a valid software upgrade for EPOC Host and Reader

Selected file: C:\Temp\epocHostUpgrade_v2.4.15
File Created on: 2008-11-06 06:33:13
EPOC Host SW version: 2.4.15
EPOC Reader SW version: 2.1.0.7
3. To schedule the file described above to be sent to all EPOC Hosts, press "Upload" button below:

No previous upgrade found
File Created on:
EPOC Host SW version:
EPOC Reader SW version:

6. **VERIFY** that the file contains the correct Host and Reader software version to be installed.
7. If the Host/Reader software versions are correct, click **Accept** to accept the software.

epoc[™] EDM EPOC Data Manager
Connected to : epocDemo User: epocsysadmin EPOC Link @ 192.168.20.121 : 12345 ✓ DB : ✓ 2.1.5 [Log Off](#)
EPOC Manager > Host Settings > Software Update
Blood Tests QA Tests EPOC System
Host Settings - Software Update
1. Use the "Browse" button below to select software load provided by Epocal's support services:

2. Press the "Verify" button to validate that the file contains a valid software upgrade for EPOC Host and Reader

Selected file: C:\Temp\epocHostUpgrade_v2.4.15
File Created on:
EPOC Host SW version:
EPOC Reader SW version:
3. To schedule the file described above to be sent to all EPOC Hosts, press "Upload" button below:

Last upgrade file uploaded: 08-Dec-08 12:21
File Created on: 11/6/2008 6:33:13 AM
EPOC Host SW version: 2.4.15
EPOC Reader SW version: 2.1.0.7


8. **The EDM is now ready to upgrade epoc Hosts.**

How to Upgrade the epoc Host (Version 2.5.8 or higher)


1. Log in to the Host as **Administrator**.




2. Select **"Tools→Perform upgrade→From EDM"**.

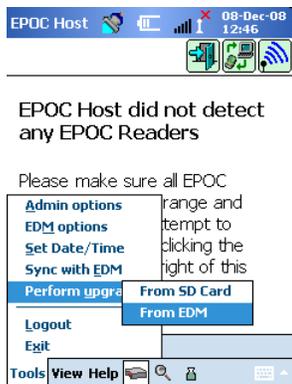

Select **"Tools→Perform upgrade→From EDM"**

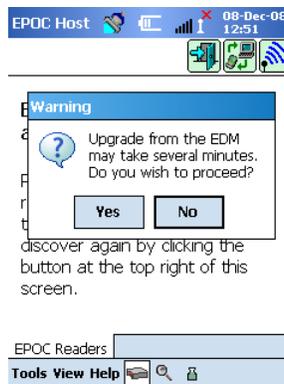


Tap on **"Yes"** to perform upgrade.

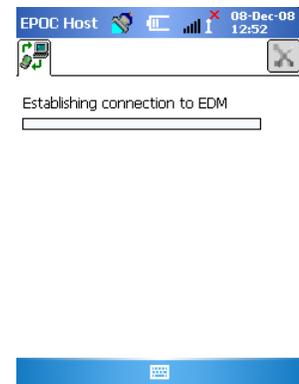


Retrieving Host software update.



Host update received...

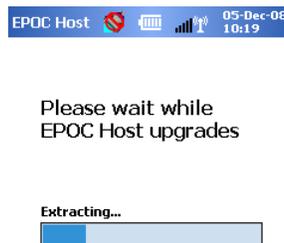

Processing host update...

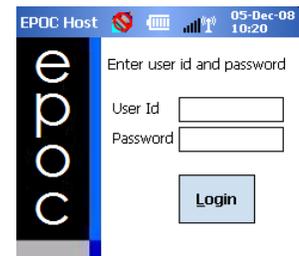

Upgrade complete. Verify version.


3. Verify the Host software version at the bottom right of the Login screen or log in and select **"Help→About"**.
4. The epoc Host is now ready for use.

How to Upgrade the epoc Host (versions before 2.5.8)


1. Log in to the Host as **Administrator**.




2. Select **"Tools→Sync with EDM"**.

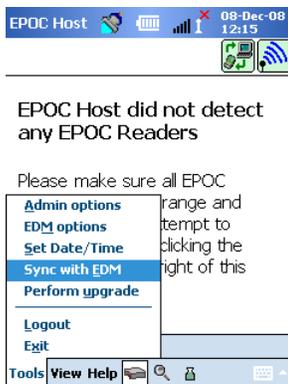


Select **"Tools→Sync with EDM"**

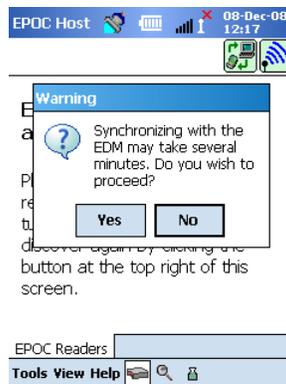


Select **"Yes"** to synchronize with EDM.

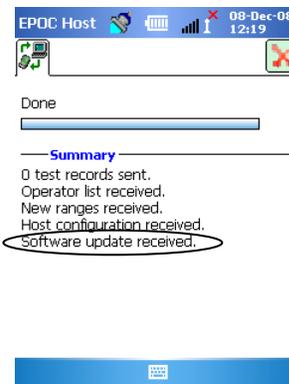


Synchronization complete. Tap  to exit.

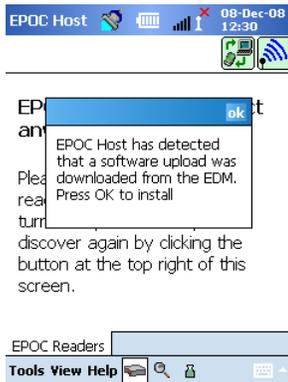

Tap **"ok"** to install update.

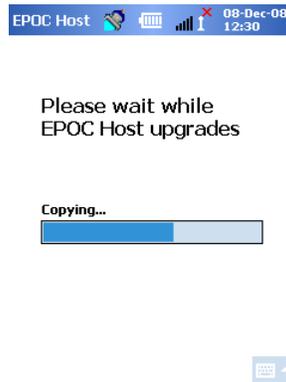

Processing Host update.

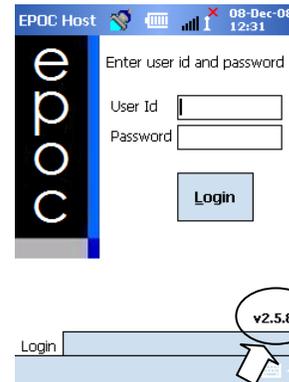

Update complete. Verify new version.

3. Verify the Host software version at the bottom right of the Login screen or log in and select **"Help→About"**.
4. The epoc Host is now ready for use.