

Overview


These instructions describe how to upgrade the ePOC Reader firmware using the ePOC Host.

How to Upgrade the epoc Reader firmware.


1. Plug in the ePOC Reader and turn it ON.
2. Log in to the ePOC Host as **Administrator**.




3. Allow the Host to discover the Reader and navigate to the Reader's upgrade tab.

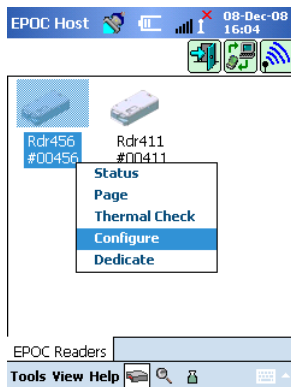

Tap and hold Reader icon and select "Configure" from the menu

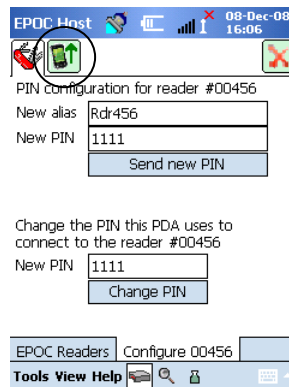


Select the upgrade tab ().

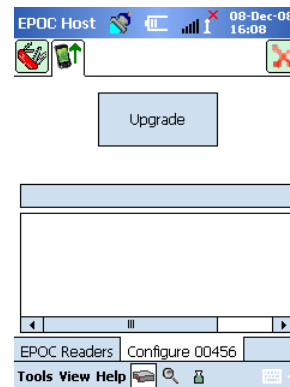


Tap on the "Upgrade" button.



Host connecting to Reader...

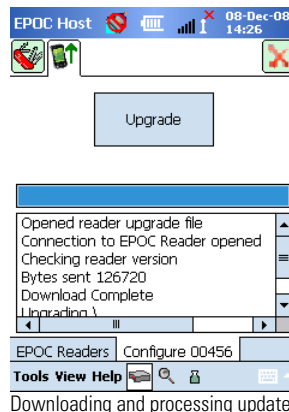


Downloading and processing update...

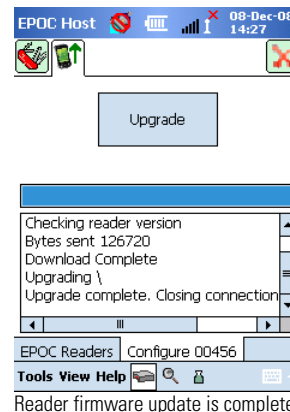


Reader firmware update is complete.

Tap the icon to exit


4. Verify that the correct Reader firmware version was loaded successfully.

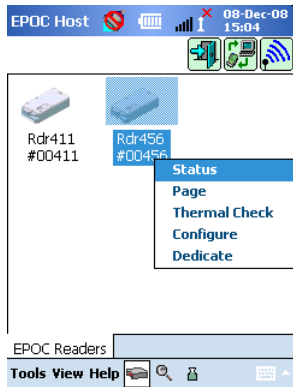

Tap and hold Reader icon and select **"Status"** from the menu

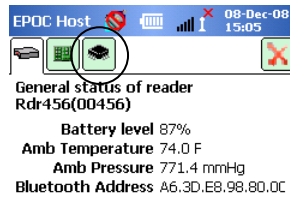

Select the configuration tab ().

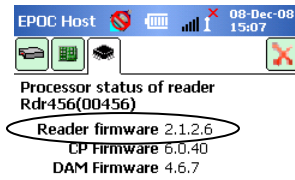

Verify Reader firmware version.

5. The epoc Reader is now ready for use.