

Overview

These instructions describe how to perform a Host-to-Host software upgrade.


What you need

This procedure requires two epoc Hosts.


1. **Host #1** contains the newest version of epoc Host software:
2. **Host #2** is the host you wish to upgrade.

How to Perform a Host-to-Host Software Upgrade

1. Turn ON BOTH **Host #1** and **Host #2**.
2. Log into the Host as administrator.
User Id: **administrator**
Password: **administrator**
3. Set EDM present to "**No**" on both Host #1 and Host #2.

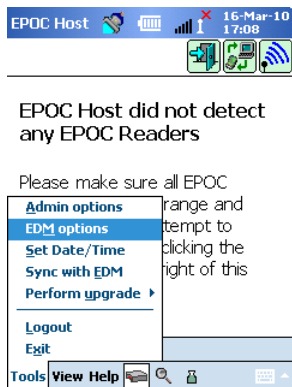


Tap "**Tools**→EDM options"

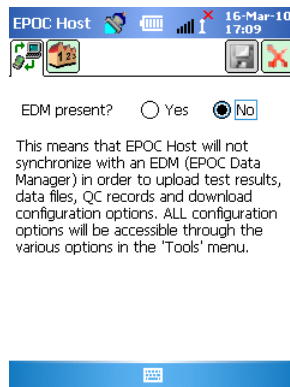


Set "**EDM present**" to "**No**"


4. On **HOST #2**, select **"Tools→Perform upgrade→From Host"**.

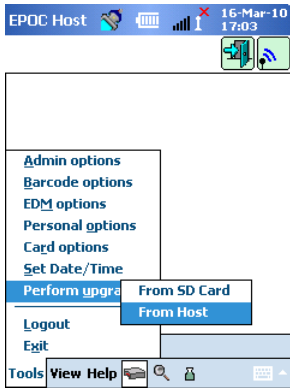


Tap **"Tools→Perform upgrade" →From Host"**

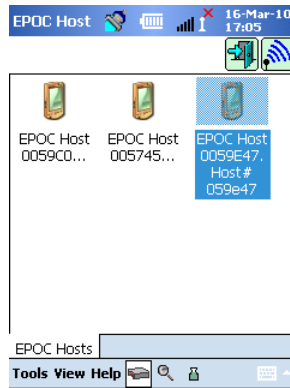


Host #2 searches for other Hosts

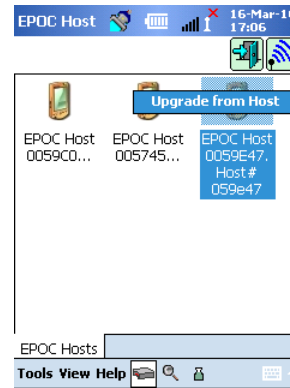


Find **Host #1** in the list. Double tap **"Upgrade from Host"**

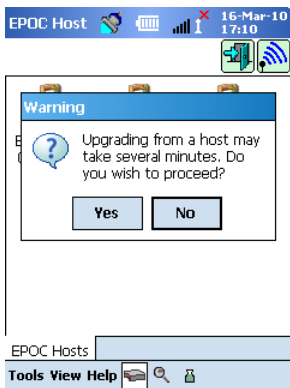


Verify that you wish to proceed.

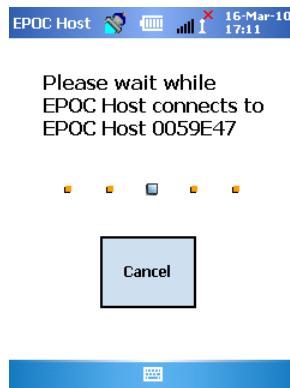


Connecting to **Host #1**...

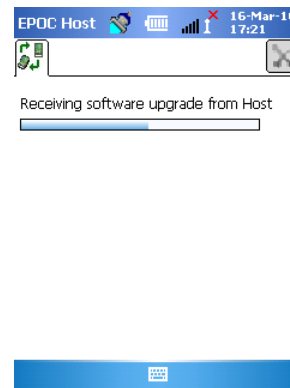


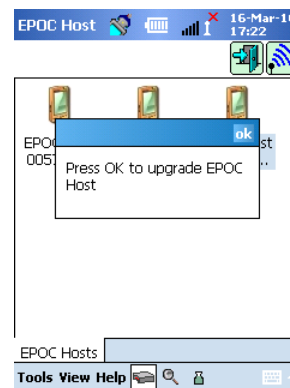
Receiving upgrade file from Host #1...

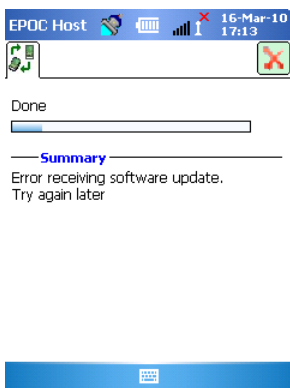

If error occurs, start over and try again.

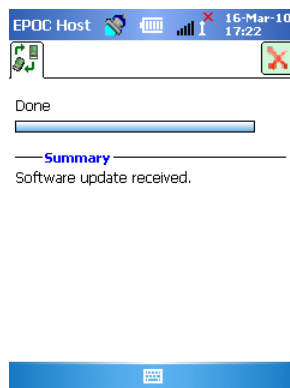

Software update received from **Host #1**
Tap on the  to install software.



Press OK to upgrade epoc Host software