


## Testing a Sample

1. Insert correct species chip (DOG or CAT) **when g-Pet is switched off**
2. Match the code number displayed on the g-Pet with the code on the strip vial
3. Remove a test strip from the vial and re-cap immediately (store at room temp)
4. Insert test strip into the test port, white side facing up
5. The species code will display. Ensure this matches code on g-Pet strip vial
6. Sample whole blood using an insulin syringe or lancet provided
7. Apply sample to tip of test strip. **Ensure the sample confirmation window is filled with blood\*\*\***
8. Result displayed in 8 seconds and stored in memory
9. Discard used strip in clinical waste

## Testing the Quality Control Solution


1. Insert a species chip **when the g-Pet is switched off**
2. Match the code number displayed on the g-Pet with the code on the strip vial
3. Insert test strip into the test port, white side facing up
4. The species code will display. Ensure this matches code on g-Pet strip vial
5. Press S button for 3 seconds until  symbol appears
6. Mix control solution bottle then discard first drop
7. Squeeze a small drop of solution onto a clean surface and apply to tip of test strip. **Ensure the sample confirmation window is filled with blood\*\*\***
8. Result displayed in 8 seconds and stored in memory
9. Compare result to specified range on test strip vial
10. Discard used strip in clinical waste

### When do you test the quality control?

When using the g-Pet for the first time  
 On a weekly basis to assess the performance of the g-Pet  
 When using a new batch of test strips

### What if the control results are outside the normal range?

Has the test been performed correctly?  
 Has the control solution expired or been stored incorrectly?  
 Have the test strips expired or been stored incorrectly?  
 Has the g-Pet been coded correctly?

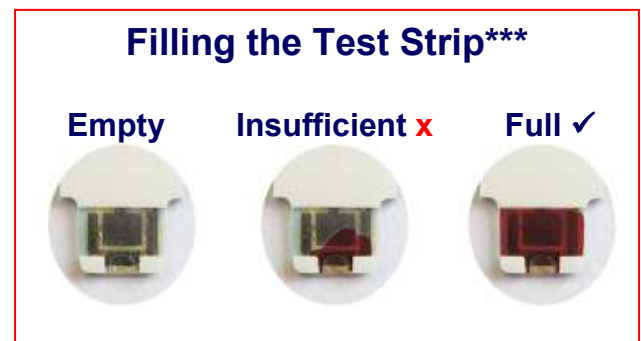

If you require assistance please contact your Woodley Equipment Area Account Manager or Woodley Equipment Company Technical Support Dept. on 01204 669033 option 1.

<b>Tel: +44 (0)1204 669033</b> <b>Fax: +44 (0)1204 669034</b> Email: <a href="mailto:sales@woodleyequipment.com">sales@woodleyequipment.com</a> Web <a href="http://www.woodleyequipment.com">www.woodleyequipment.com</a>	<b>WOODLEY</b> <b>EQUIPMENT COMPANY LTD.</b>
---	---

OLD STATION PARK BUILDINGS . ST. JOHNS STREET . HORWICH . BOLTON . LANCASHIRE . BL6 7NY